GB PROCU LIN[®] SOFT LENS MULTIPURPOSE SOLUTION with hyaluronate acid, for soft contact lenses, disinfecting, rinsing, storing and moistening. For soft contact lenses.

MEDICAL DEVICE

Composition:
Active Ingredient: Polyquaternium, Hyaluronate
Preservative: PHMB 0.0002%
Read the instructions and warnings carefully for proper use of the PROCU LIN[®] SOFT LENS in combination with your contact lenses. Do not use PROCU LIN[®] SOFT LENS if you are allergic to one of the ingredients.
Contains sodium hyaluronate that enhances the moistening of lenses and in case the tolerability of lens-wear due to its viscous-elastic properties. The solution hydrates lenses due to its water-lock, not only providing moisture but also ensuring a comfortable wearing comfort of the lens and diminishes eye irritation and dryness.

- Precautions and instructions for use:**
- Wash your hands carefully before removing the lenses from your eyes or the lens case. Dry your hands using a clean towel.
 - Unscrew the cap from the bottle while breaking the guarantee seal. After opening the bottle, keep the cap closed. Do not use the eye drops PROCU LIN[®] SOFT LENS. Clean the lenses by carefully rubbing them with your index finger in the palm of your other hand for 10-15 seconds.
 - Do not use PROCU LIN[®] SOFT LENS.
 - To clean the lenses, place them in the lens case filled with fresh PROCU LIN[®] SOFT LENS and leave it to stand for at least 6 hours. Remove the lenses from the lens case after the disinfecting cycle, rinse them with PROCU LIN[®] SOFT LENS and insert the lenses into your eyes.
 - Rinse the lens case with PROCU LIN[®] SOFT LENS and allow the lens case to dry in the air.
 - Close the bottle tightly immediately after use.

Other instructions:
After storage in excess of 7 days, lenses must be cleaned and disinfected again before wear.
Replace your lens case frequently to prevent infection due to contamination.

- Warnings:** Use if guarantee seal is damaged.
- Do not mix or use PROCU LIN[®] SOFT with other contact lens fluids, unless it is indicated in the instructions for use.
 - Always replace any remaining solution in the lens case with fresh PROCU LIN[®] SOFT LENS.
 - Do not use after the expiry date.
 - Do not change your contact lens system without consulting your eye care professional.
 - Never use tap water as an alternative for this contact lens fluid.

Do not use:
Do not use PROCU LIN[®] SOFT LENS if you are allergic to one of the ingredients.
Do not use PROCU LIN[®] SOFT LENS if you are allergic to one of the ingredients.
Store between 4°C and 25°C (40°F and 77°F).
Protect from sunlight.
Keep the solution in its original packaging.
Use within 3 months of opening.

Side Effects:
Do not use PROCU LIN[®] SOFT LENS if you are allergic to one of the ingredients.
Store between 4°C and 25°C (40°F and 77°F).
Protect from sunlight.
Keep the solution in its original packaging.
Use within 3 months of opening.

МК PROCU LIN[®] SOFT LENS POBEKHAHEMCKИ PACTBOP CO HИAЛOHAHO KИCЛOТИHO. ДИCИHEKТИO, ПPOCИYBИO, XPAHEHИE И MOИCТEHИE ПO MЯKOY K OHTAKT HИE. ДИCИHEKТИO, ПPOCИYBИO И MOИCТEHИE ПO MЯKOY K OHTAKT HИE.

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